



RECOVERY CARE PRODUCTS

THE GIFT OF RECOVERY

SCAR MANAGEMENT

100% Silicone Gel

For New and Existing Scars



Indicated for Topical Management of Keloids & Hypertrophic Scars

- Prevention & Management of Abnormal Scarring
- Reduces Itch, Redness and Discoloration
- Softens and Flattens Raised Scars
- Effective on 1st and 2nd Degree Burns
- FDA Cleared
- Made in the USA in an FDA Compliant Facility

Ingredients

- Dimethicone, Dimethicone / Vinyl dimethicone Crosspolymer

Directions

- Please adhere to your physician's protocol
- Begin using when stitches are removed (or dissolved) and when the wound is fully closed with no scabbing (typically 2-3 weeks after surgery)
- Wash the affected area with a mild soap such as **RCP Gentle Cleanser** and pat dry
- **MORE IS NOT BETTER!** A little silicone gel goes a long way.
- Apply roughly a pea size amount for a 2"x4" area and massage onto the skin.
- For smaller or larger areas adjust accordingly
- If application feels excessively tacky or thick, you've applied too much. Simply wipe away the excess until dry
- Reapplication may be needed when used on areas that require frequent washing or are exposed to excessive rubbing
- Once dry apply **RCP Mineral Sunscreen** over the silicone gel
- Use consistently morning and night for a minimum of 12 weeks
- If you are prone to hypertrophic scars or keloids you may use safely for 6 months or longer as needed

Warnings

For external use only. Avoid using near the eyes or mucus membranes. If redness or irritation occurs discontinue use and consult your physician. Avoid use on deep or puncture wounds, animal bites or serious burns. Excess application may stain clothing. **Keep out of reach of children.** Protect this product from excessive heat and direct sunlight.

net wt 0.5 oz / 15 g